

This document contains important information for safe and proper handling of your type B, VS and S Siroclaves and Siromini

Safety notice due to a potential incident

Addressees: All system owners of type B, VS and S Siroclaves and Siromini
Sender: Sirona Dental Systems GmbH, Fabrikstrasse 31, 64625 Bensheim
Subject: Possible breakage of the door latch locking bolt

Please make sure that all of your employees who use the above-referenced system are informed about this safety notice and file it in the operating instructions pertaining to your system. If you are no longer in possession of the system, please forward this safety notice to the new owner if you know his correct address.

Problem description:

The locking bolt of the door latch can bend or even break under unfavorable conditions.

Possible hazard:

If the locking bolt of the door latch breaks, the locking mechanism of the chamber may burst off due to the excess pressure in the sterilizer.

What should you do to prevent this from happening?

If your unit has performed more than 10,000 cycles or is more than four years old, contact your dental retail specialist immediately to schedule the required maintenance.

What may happen if this notice is disregarded?

Failure to perform the required maintenance may result in a potential hazard for anyone located in the immediate vicinity of the unit.

Recommended actions to be taken by the operator:

- Determine the number of operating cycles and service life of the unit
- Contact your dental retail specialist
- Observe the required maintenance intervals

Measures taken by Sirona Dental Systems GmbH:

Specification of required maintenance measures

For more information or assistance, please contact the Sirona Hotline, telephone +49 6251-16-180, or e-mail product.service@sirona.com